


Advanced Controller (Red LED display)

To View Min/Max 'Air' Temperature

The controller arrives to with 'running air temperature' as default




STEP 1

For MAXIMUM 'air' temperature press  and hold for 3 seconds and the MAXIMUM value flashes





STEP 2

For MINIMUM 'air' temperature press  and the MINIMUM value flashes



STEP 3

To reset both values

Press and hold  then press and hold  and hold both for 3 seconds (whilst viewing the min or max value)



STEP 4

Your min/max has been reset

If You Prefer to Record Load/Vaccine Temperature



STEP 1

To view 'load' temperature press and hold **MENU** until "LOA" is displayed and then hold for 3 seconds



STEP 2

To view **RUNNING** load temperature Press **←** and hold for 1 second



STEP 3

To view **MAXIMUM** load temperature Press **↑** and hold for 1 second



STEP 4

To view **MINIMUM** load temperature Press **✓** and hold for 1 second



STEP 5

To **reset both values** Press and hold **↑** then press and hold **✓** and hold both for 3 seconds (whilst viewing the min or max value)



STEP 6

Your min/max has been reset. Controller will relax to normal display after approximately 30 seconds